



**The First Aid
Kit in a Tube**



**For Use On ALL
Animals Including
Birds & Reptiles**

**NET WEIGHT:
1 OZ. / 28 g**

LIMITED WARRANTY AND DISCLAIMER

The manufacturer warrants only that the chemical composition of this product conforms to the ingredient statement given on the label and that the product is reasonably suited for the labeled use when applied according to the Directions for Use.

THE MANUFACTURER NEITHER MAKES NOR INTENDS ANY OTHER EXPRESS OR IMPLIED WARRANTIES, INCLUDING ANY WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, WHICH ARE EXPRESSLY DISCLAIMED. This limited warranty does not extend to the use of the product inconsistent with label instructions, warnings or cautions. Any damages arising from a breach of the manufacturer's warranty shall be limited to direct damages, and shall not include indirect or consequential damages such as loss of profits or values, except as otherwise provided by law.

The terms of this Limited Warranty and Disclaimer cannot be varied by any written or verbal statements or agreements. No employee or agent of the seller is authorized to vary or exceed the terms of this Limited Warranty and Disclaimer in any manner.

**For more information call
1-800-336-7087
or visit www.emtgel.com**

104/8-2004

INGREDIENTS:

Type I hydrolyzed collagen
Purified Water, U.S.P.

MANUFACTURED FOR
TROPHY
ANIMAL HEALTH CARE
PENSACOLA, FLORIDA 32514
www.trophyanimalcare.com

READ CAREFULLY BEFORE USING

EMT Gel is a revolutionary breakthrough in wound care for animals. EMT Gel provides nutritive collagen protein to the wound site to enhance the natural healing process and promote rapid healing. EMT Gel reduces bleeding by forming a plug to encourage natural clotting. EMT Gel seals off nerve endings to reduce pain and itching. EMT Gel also forms a semi-occlusive barrier that protects the wound and provides a moist environment which encourages healing. EMT Gel is all-natural, non-toxic, and has no known side effects.

INDICATIONS —

For topical application on all animals in the management of chronic and acute wounds such as:

- Superficial wounds (cuts, abrasions)
- Burns
- Traumatic wounds (flesh tears)
- Dermal ulcers; 'hot spots'
- Injured foot pads
- Surgical wounds
- Other general dermatologic conditions

CHARACTERISTICS —

EMT Gel reduces the chance of infection by forming a barrier over the wound site. Easy to apply, EMT Gel:

- Promotes Rapid Healing
- Reduces Bleeding
- Protects Wounds
- Reduces Pain and Itching
- Conforms to any Wound Site
- Soothes and Deodorizes
- Absorbs Wound Fluids
- Biocompatible, Biodegradable

HOW TO USE —

1. Cleanse wound thoroughly with water or saline.
2. Apply EMT Gel directly into the wound and onto the surrounding area.
3. If needed, cover wound with a non-stick dressing (Telfa®). Allow to air dry on superficial wounds.
4. Reapply EMT Gel and redress daily (if needed).

STORAGE —

Store at controlled room temperature. Extreme heat may affect viscosity, but will not affect product performance.

Protect from freezing.

PRECAUTIONS —

Keep out of reach of children.

If condition worsens, or does not improve within 10-14 days, consult a veterinarian.



Telfa® is a registered trademark of
The Kendall Company.